4WAXclean

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EAR hygiene SPRAY

INSTRUCTIONS FOR USE

Please read the instructions for use carefully. They contain important information for the patient about the application of the medical device. Please keep the instructions for further reference, if needed.

1. WHAT IS THE 4WAXCLEAN EAR HYGIENE SPRAY AND WHAT IS IT USED FOR?

4WAXclean ear hygiene spray is an effective, safe and simple method to soften and naturally remove ear wax and prevent water buildup in the ear while swimming (so-called "swimmer's ear"). Using 4WAXclean ear hygiene spray helps gradually breakdown wax and remove it naturally from the ear. It reduces the necessity for surgical earwax removal by a doctor. Earwax is a secretion of glands in the ear. Excessive secretion can create deposits (plugs, tenons) of wax and cause ear pain, discomfort and in some situations deterioration of hearing, including its loss.

Ear infections can result from bacterial growth due to lingering water. 4WAXclean ear hygiene spray protects the skin inside the ear, preventing water accumulation. The medical device 4WAXclean ear hygiene spray can be applied while standing, sitting or lying and disperses well in the ear canal.

2. IMPORTANT INFORMATION BEFORE USING THE MEDICAL DEVICE 4WAXCLEAN EAR HYGIENE SPRAY

Do not use the medical device 4WAXclean ear hygiene spray:

- in case of previously known hypersensitivity to any of the components of the solution,
- in case eardrum perforation is found or suspected,
- in case of presence or suspicion of a foreign body in the ear,
- if there is ear leakage,
- without consulting a doctor in children under 3 months of age,
- without consulting a doctor in case of earache accompanied by high body temperature,
- in case of hearing diseases,
- into other body orifices, e.g., eye, nose.

Warnings and Precautions:

Pregnant women should obtain medical advice before using the product. Always consult your doctor if you experience recurrent and unexplained outer ear symptoms.

3. HOW TO USE THE MEDICAL DEVICE 4WAXCLEAN EAR HYGIENE SPRAY

Directions for use:

Medical device intended for administration into the ear.

Instructions for use of the medical device 4WAXclean ear hygiene spray:

1. Before initial usage, place the applicator in a horizontal position and press the dispenser 3-4 times to fill it with liquid.
2. Gently pull the upper earlobe up and to the back of the head.
3. Holding the bottle upright, insert the applicator into the external ear canal and firmly press the top of the dispenser.
4. After application, massage the front of the ear canal (scrape) from the outside.
5. If required, repeat steps 2 and 3 for the other ear.
6. After usage, clean the dispensing tip with warm water and dry.
7. Turn the applicator vertically to prevent accidentally spraying the fluid.

Dosage:

Children over 3 months to 12 years of age:

- To remove wax buildup: 1 dose 2-3 times daily to each ear until wax is dissolved;
- To prevent earwax collection: 1 dose once a week;
- As a preventive measure before swimming: 1 dose in each ear.

Use under adult supervision. Do not insert the applicator too deeply into the ear to prevent hearing impairment.

Children over 12 years of age and adults:

- To remove wax buildup: 1-2 doses 3 times daily to each ear until wax is dissolved;
 - To prevent earwax collection: 1 dose 1-2 times per week;
 - As a preventive measure before swimming: 1 dose in each ear.
- If you have any concerns about the effectiveness of the product, consult with your doctor or pharmacist.

Use of a higher than recommended dose of the medical device 4WAXclean ear hygiene spray.

Do not exceed the suggested doses without consulting your doctor.

Missing the application of 4WAXclean ear hygiene spray.

Skipping a dose of the product has no significant effect on the course of treatment.

Continue treatment at the current dosage.

Do not double the dosage to compensate for a missed dose.

4. POSSIBLE SIDE EFFECTS

If you experience allergic reactions or other adverse reactions, discontinue use and consult a doctor if necessary. All adverse reactions should be reported to the manufacturer.

5. HOW TO STORE 4WAXCLEAN EAR HYGIENE SPRAY

Store at 15-25°C in closed, original packaging. Keep out of the reach and sight of children. Once opened, use within 6 months. Do not use medical device 4WAXclean ear hygiene spray after the expiration date indicated on the packaging.

6. COMPOSITION OF THE MEDICAL DEVICE 4WAXCLEAN EAR HYGIENE SPRAY

Olive oil, almond oil, jojoba oil, liquid paraffin, vitamin E, polysorbate 80.

7. PACKAGE DESCRIPTION

4WAXclean ear hygiene spray is packaged in a bottle containing 20 ml of liquid. The bottle is equipped with a dispenser. The unit pack includes: a cardboard box, a 20 ml bottle with liquid and instructions for use.

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