HYPERTONIC SEA WATER nasal spray

INSTRUCTION FOR USE

Please read the instructions for use carefully, as they contain important information for the patient about the application of the medical device **HYPERTONIC SEA WATER**. Please keep the instructions for further reference, if needed. Product contains sea water collected from the depths of the Atlantic Ocean, rich in natural minerals and is intended for nasal congestion caused by cold, hay fever or allergies.

COMPOSITION:

Natural sea water, demineralized water. Salt concentration 2,2%.

INDICATIONS:

It is recommended for use in adults, children and infants from 1 month of age:

- to clear the nose in case of rhinitis, allergies, chronic sinusitis, cold,
- to relieve congestion of the nasal mucosa (during a cold, inflammation of the mucosa of the nose and throat, allergic rhinitis, inflammation of the paranasal sinuses),
- to improve the condition of the mucosa (in case of chronic rhinitis, sinusitis),
- to thin out the secretions and clean the nasal mucosa, which facilitates the effects of nasally administered drugs and reduces the risk of bacterial superinfection,
- to cleanse the nose, which helps to protect the mucosa from external factors and helps to provide nasal breathing functions.

DOSAGE:

Children from 1 month of age and adults: 1 to 2 doses of spray in each nostril at intervals of 4-5 hours. For infants from 1 to 3 months of age, indirect application with a cotton pad is recommended. If you experience symptoms of discomfort in the nasal cavity, apply 1 dose to the nasal cavity 3 times a day. Use in children under 3 years of age should be preceded by consultation with a pediatrician. In case of doubts, consult your doctor or pharmacist again. This product does not affect your ability to drive and use machines.

USE OF A HIGHER THAN RECOMMENDED DOSE OF THE MEDICAL DEVICE: A higher dose of the medical device does not pose a threat to children over 1 year of age. Observe the recommended usage. The product is intended for continuous use for no more than 30 days.

DIRECTIONS FOR USE:

Wash your hands before application. Remove the protective cap and press the applicator several times until an even spray is produced (Fig. 1). Gently insert the applicator into the nose and press the applicator base (Fig. 2). After each use, clean the tip with water and soap, then rinse and dry. Replace the protective cap.

PRECAUTIONS AND STORAGE CONDITIONS:

To avoid the spread of infection, it is recommended that one product should be used by one person. Avoid direct sunlight. Do not use the product if you are hypersensitive to any of its components. Do not use after the expiry date. The lot number and expiry date: on the bottle and outer box. Store in the original packaging at temperatures between 5°C and 25°C. **STOP USING THE PRODUCT** if you experience itching or burning sensation after using the product, which may indicate that you are allergic to components of the product. Do not use with other nasal preparations without consulting a doctor. Do not use if the package is damaged or it has been tampered with before first opening. The product is intended for pediatric use. When used in children, it requires the action of a parent / guardian delicate enough not to damage the nasal mucosa. The product is intended for nasal cavity use only.

KEEP OUT OF THE REACH AND SIGHT OF CHILDREN. DO NOT USE IN PREMATURE INFANTS.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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